


Hippo-U


The advantages of Hippo-D remain the same, but we have significantly lowered the price

Enhanced retention with completely different materials

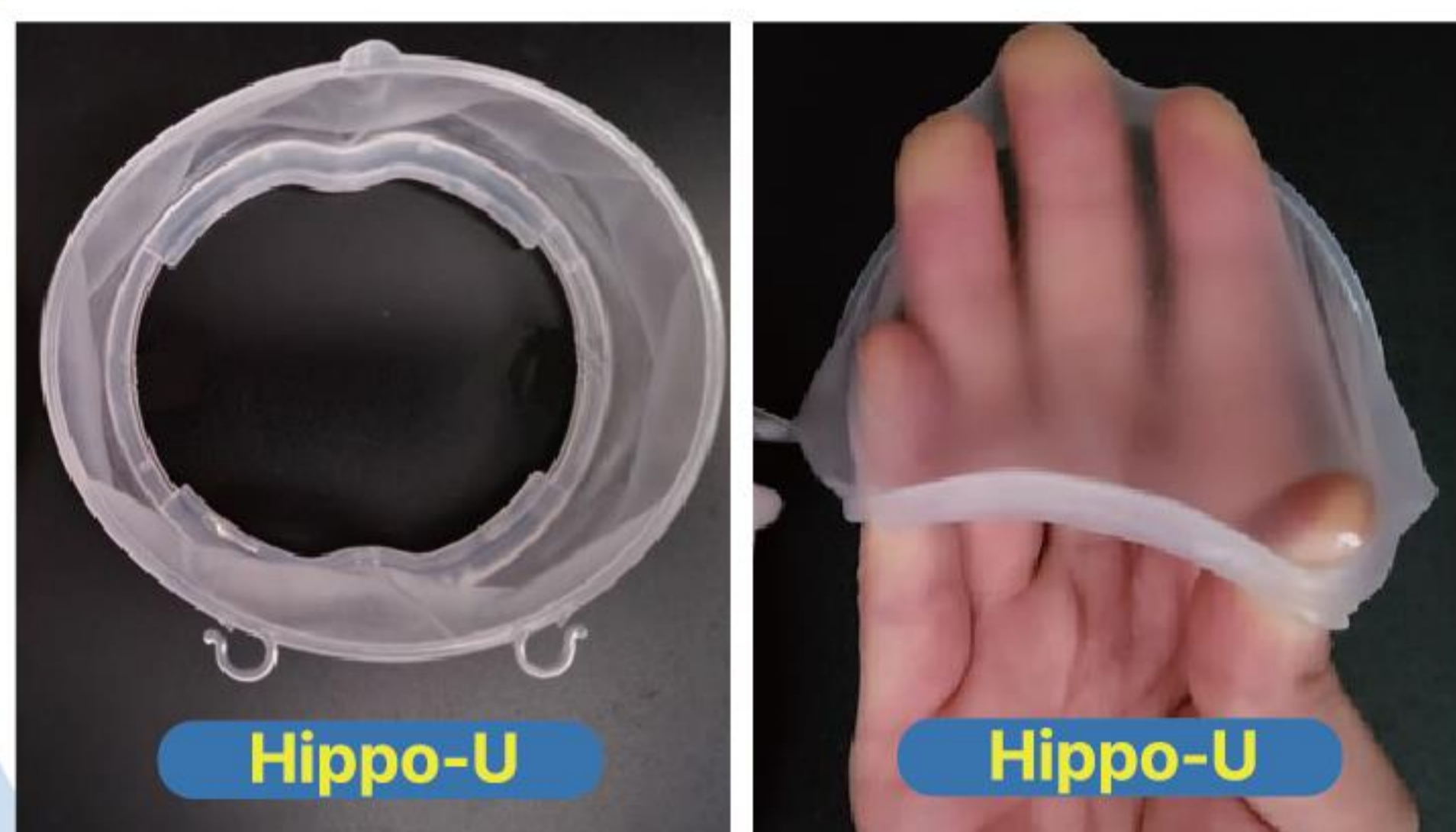

Hippo-U advantage

- ✓ We reduced thickness while increasing elasticity
- ✓ Remarkable improvement in retention, enabling wear with easy opening and closing
- ✓ Suitable for over 95% of users with just one size
- ✓ Extraction of soft tissue possible without the use of a left-handed dentist
- ✓ Providing patient comfort and preventing splatter on the face
- ✓ External rings can be fixed with suction tips
- ✓ Wearing condition can be checked through center

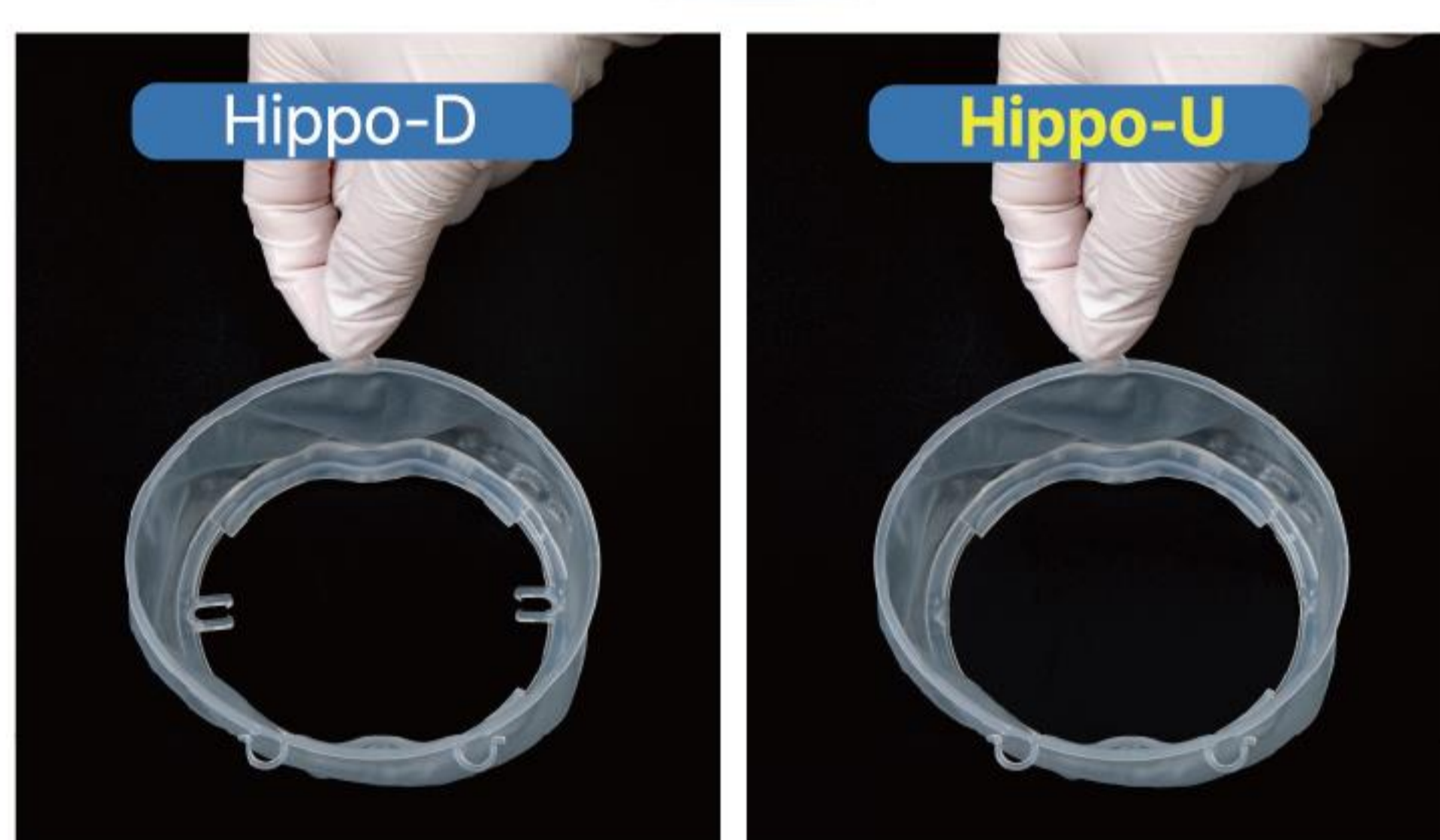

Hippo-D VS Hippo-U

- ✓ Same material
- ✓ No internal rings
- ✓ Can be used without the Hippo D-line

How to wear Hippo-U



Spray water



Put on the patient's right side, then the left side



Arrange the upper and lower lips



Check the wearing condition with Hippo-U center.

How to use Hippo-U

It's recommended to follow the order below

1



Spray water on the retractor before use.



2



Wearing in the order of
right → left
→ bottom → top
on the patient.



3



Adjust the upper and lower lips with a mirror so that the red part of the lips is visible.

Use cases of Hippo-U

HippoU can be used for various medical services.



ORTHODONTICS



IMPLANT



ANTERIOR REGION



**DENTAL
PROSTHETICS**



**RESTORATIVE
DENTISTRY**



EXTRACTION



WHITENING



**PERIODONTAL
TREATMENT**

Advantages of Hippo-U

The use of proprietary new material has dramatically increased retention.



Impressions and bite
can be verified
while wearing it.



It can also be used for
patients with
no natural teeth.



Using it like an umbrella
during **anterior treatments**
prevents aerosols from entering
the nose and avoids the need for
mirror retraction.